

Since 1955
GINGI-PAK®

SDFPRO™

38% SILVER DIAMINE FLUORIDE

**Precise Dispensing
with Flip-Top Cap**



**Relieves
HyperSensitivity**



**Noninvasive
Treatment**



Since 1955
GINGI-PAK®

4825 Calle Alto, Camarillo, CA 93012 | (800)437-1514 | info@gingi-pak.com | www.gingi-pak.com | Made in USA

SDF PRO™

38% SILVER DIAMINE FLUORIDE

Gingi-Pak's SDF Pro 38% Silver Diamine Fluoride solution is formulated to reduce dentinal hypersensitivity and provide primary prevention on high-risk sites. It penetrates healthy dentin and enamel without causing discoloration, delivering long-lasting antimicrobial protection while improving acid and abrasion resistance of compromised dentin.

Ideal for challenging situations where traditional restorative treatments like fillings or sealants are not possible, or in identifying hidden lesions, SDF Pro 38% offers a non-invasive alternative that avoids the need for drills, needles, or anesthetic, creating a more comfortable patient experience.


Compare to Advantage Arrest® Silver Diamine Fluoride

Key Features

- Reduces dentinal hypersensitivity and kills microorganisms
- Enhances acid and abrasion resistance of compromised dentin
- Non-invasive-no drills, needles, or anesthetic needed



Comparison of SDF Products by Components

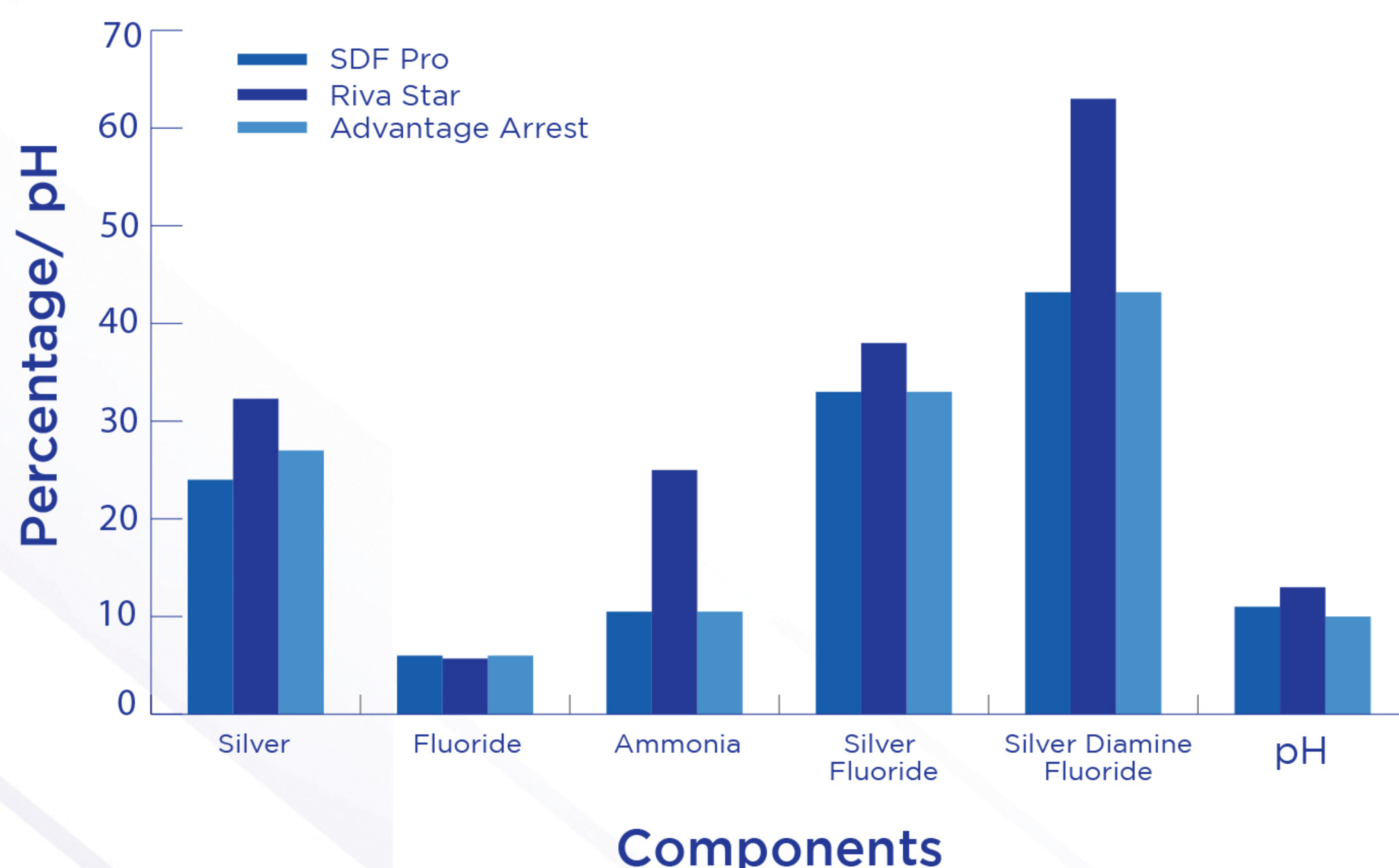

- SDF Pro's moderate pH indicates increased stability and patient comfort
- SDF Pro's lower ammonia level indicates reduced odor and increased comfort


**data on file*

SDFPRO™

38% SILVER DIAMINE FLUORIDE

4 Easy Steps

STEP 1

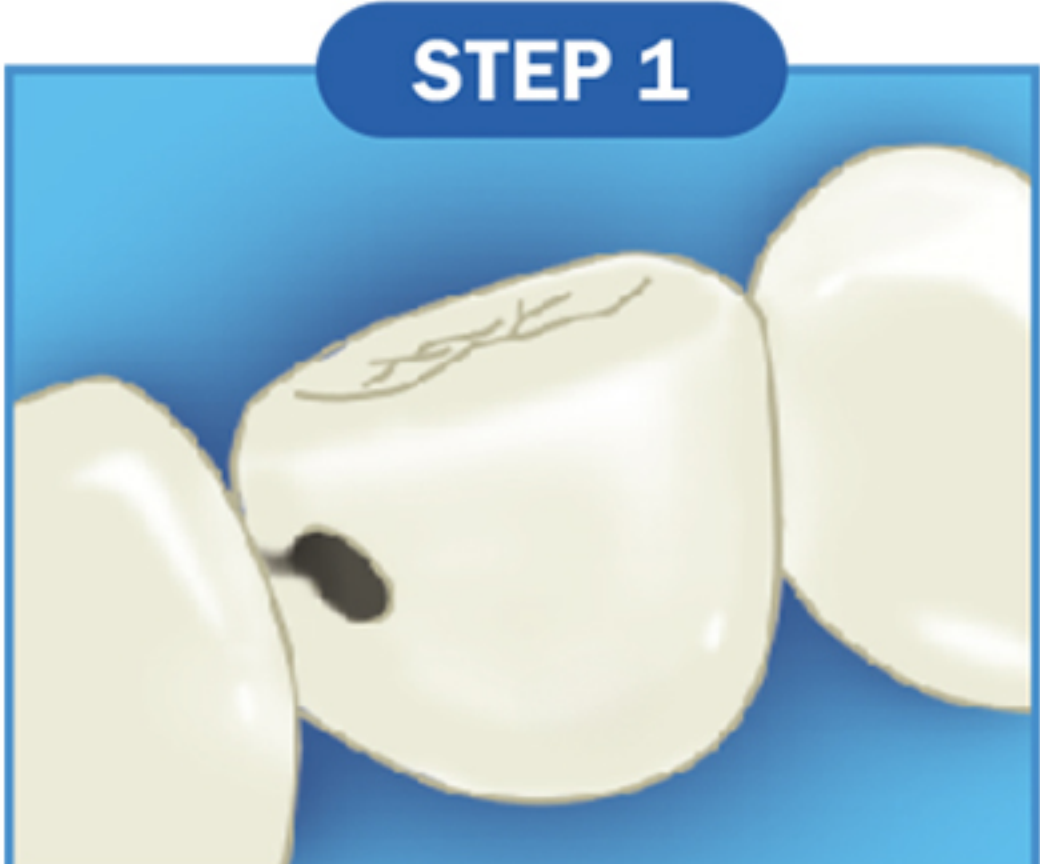

Remove excess saliva and isolate the area with cotton or gauze. Apply petroleum jelly if near the gingiva.

STEP 2



Thoroughly dry the affected tooth surface(s).

STEP 3



Dispense 1 drop of SDF Pro into a dappen dish and apply with an applicator.

STEP 4



Allow SDF Pro to absorb for up to one minute. Air dry to remove any excess saliva and precipitation.

Faster, Simpler Treatment

SDF Pro treatment offers a quicker, easier alternative to traditional invasive procedures. By following the principles of Minimal Intervention Dentistry, SDF Pro focuses on prevention, remineralization, and less invasive care — helping patients receive effective treatment with minimal discomfort or complexity.

	SDF PRO	Invasive treatment
Quick & efficient	✓	✗
Easy & simple to use	✓	✗
Cost effective intervention	✓	✗
No anesthetic required	✓	✗
No drilling required	✓	✗
No restoration required	✓	✗
No polishing required	✓	✗
No post procedure sensitivity	✓	✗



REF 73013

SDF Pro 38% Silver Diamine Fluoride Solution Unit Dose Kit:

- 30x 0.1 mL Unit Doses
- 50x Micro Applicator Tips



REF 73014

SDF Pro 38% Silver Diamine Fluoride Solution Bottle Kit:

- 1x 5 mL Bottle
- 50x Micro Applicator Tips

BUY 3, GET 1 OF THE SAME FREE!

To qualify and receive your promotional goods please submit a valid proof of purchase to info@gingi-pak.com. Invoices must be submitted via email to qualify. Cannot Be Combined With Any Other Offer. No Cash Value. Same or Lesser Item(s) Applied. Offers are valid only at participating dealers in the continental US, excluding AK and HI. Promotions are subject to change or cancellation without notice. Please allow 6-8 weeks for delivery.

Treatment Codes

- D9910 (desensitizing)
- CPT Code - 0792T (Application of silver diamine fluoride 38%, by a physician or other qualified health care professional)

Scan for FREE Sample

